

INSTRUCTIONS FOR USE  
(BOTTLE)

Riva Star is a new generation desensitizer that immediately relieves tooth sensitivity.

**INDICATIONS FOR USE:**

- Treatment of dentinal hypersensitivity.
- For use in adults over the age of 21.

**CONTRA-INDICATIONS**

- Do not use in patients with ulcerative gingivitis, stomatitis or patients undergoing thyroid gland therapy.
- Do not use on patients with known sensitivity to silver or other heavy - metal ions, fluoride, potassium, iodine or ammonia. **Note:** If in doubt, consult a doctor or an allergy specialist before use.

**PROCEDURE:**

1. Using a non-fluoride prophylaxis paste in a rubber cup, clean the tooth/teeth to be treated and abutting teeth.
2. Assess condition of teeth. Only teeth with sound dentine and enamel should be treated.
3. Isolate area to be treated using cotton rolls and protect the gingival tissues with petroleum jelly/cocoa butter taking care not to contaminate treatment site. Alternatively, a rubber dam or a light curable gingival protection can be used.
4. Protect lips with petroleum jelly/cocoa butter.
5. Before opening cap, tap the bottom of the bottle on a hard surface to minimize the liquid from coming out of the dropper tip.
6. **Riva Star Step 1:** Dispense one (1) drop of solution onto non-absorbent mixing pad.
7. Carefully apply solution to treatment site only, a medium sized micro brush, such as Points (SDI Limited) is recommended.
8. **Riva Star Step 2:** Immediately after, dispense two (2) drops of solution onto fresh non-absorbent mixing pad.
9. Apply a generous amount of the solution to treatment site, a medium sized micro brush, such as Points (SDI Limited) is recommended. Apply same solution to any other tooth surfaces that may have come into contact with the solution from the **gray** label bottle. **Note:** Treatment surface initially appears creamy white, keep applying Riva Star Step 2 solution until it turns clear.
10. Blot dry.
11. Remove all protective/isolation materials used in the mouth.
12. Discard used mixing pads and brushes in accordance with local regulations.

**DOSAGE:**

- One (1) drop of Riva Star Step 1 and two (2) drops of Riva Star Step 2 can treat up to 5 teeth per patient.
- Limit of one (1) drop of Riva Star Step 1 and two (2) drops of Riva Star Step 2 drops per visit. If needed a second application of Riva Star may be administered after 1 week.

**PRECAUTIONS:**

- For professional use only.
- Do not repeat treatment on pregnant or lactating women.
- Due to potential staining, it is recommended that anterior teeth not be treated.
- Patient must be informed of potential staining prior to treatment.
- Avoid contact of solution from Riva Star Step 1 (**gray** label) with oral soft tissue, eyes, skin and clothing. If accidental contact occurs refer to the "FIRST AID" below.
- Avoid contact of solution from Riva Star Step 2 (**green** label) with eyes.
- Do not desensitize teeth with soft dentine as staining may occur. Note: If it does occur, stained sites can be reduced by using a judicious polishing procedure with phosphoric acid impregnated pumice powder and then covered with a self-cure glass ionomer cement (such as Riva Self-Cure, SDI Limited).
- Keep out of reach of children.
- Do not ingest.
- Solution from Riva Star Step 1 (**gray** label) will stain (see "HANDLING STAINS").
- Wear gloves and eye protection when handling.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- Solution from Riva Star Step 1 (**gray** label) contains 0.06g / mL fluoride ions.
- SDS is available at [www.sdi.com.au](http://www.sdi.com.au) or contact your SDI representative.

**STORAGE AND HANDLING:**

- Do not dispense solution from **gray** label bottle or **green** label bottle until ready to use.
- Store in a cool and dark place at temperatures between 5° and 25°C (41° - 77°F).
- Use at room temperature between 20° and 25°C (68° - 77°F).
- Replace cap on bottle and seal tightly immediately after use.

**FIRST AID:**

- **Eyes (contact):** Hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre or a doctor, or for at least 15 minutes. Seek medical attention.
- **Skin/oral soft issue (contact):** Remove contaminated clothing, flush area with running water (in case an eschar is formed, this should disappear within 24-48 hours).
- **Ingestion:** Do NOT induce vomiting. Give a glass of milk or water. Seek immediate medical attention.
- **Inhalation:** If affected, remove from contaminated area.

**HANDLING STAINS:**

- **Skin:** In case of skin contact with Riva Star Step 1 (**gray** label), apply Riva Star Set 2 (green label) with gentle rubbing. Otherwise, if Riva Star Step 2 solution is not applied, the stain should disappear gradually after a few days.
- **Clothing:** Remove contaminated clothing and soak in Napisan\* (or other fabric stain remover) overnight and wash per normal.
- **Hard surfaces:** Immediately apply household bleach (sodium hypochlorite) then wipe off with water. For stubborn stains, saturate a paper napkin with household bleach and place over affected area. Leave overnight and mop up with water.

Napisan\* is not the registered trademark of SDI Limited.

**PACKAGING:****Each kit contains:**

- 1 x Riva Star Step 1 (**gray** label) bottle 1.5 mL
- 1 x Riva Star Step 2 (**green** label) bottle 3 mL

US Patents granted.

PAK41363 B

Made in Australia by SDI Limited  
 Bayswater, Victoria 3153  
 Australia 1 800 337 003  
 Austria 00800 0225 5734  
 Brazil 0800 770 1735  
 France 00800 0225 5734  
 Germany 0800 1005759  
 Italy 00800 0225 5734  
 New Zealand 0800 734 034  
 Spain 00800 0225 5734  
 United Kingdom 00800 0225 5734  
 USA & Canada 1 800 228 5166  
[www.sdi.com.au](http://www.sdi.com.au)  
[www.rivastar.com](http://www.rivastar.com)

Rx

